



Model 4000 Hydraulic Patient Chair

1. Place the chair carton close to the planned area that you wish to install the dental chair.



2. Remove the pump cover from the front of the chair to access the two shipping screws.



3. Loosen and remove the two packing screws holding the chair to the pallet.




4. Locate the handles to place onto the chair base to assist in the lifting process.



5. Lift the chair base using the provided handles shown in the picture. Set the chair into position where you would like it located.



6. Attach the back rest mounting plate with the four bolts and lock washers that are provided with your model 4000 dental chair.

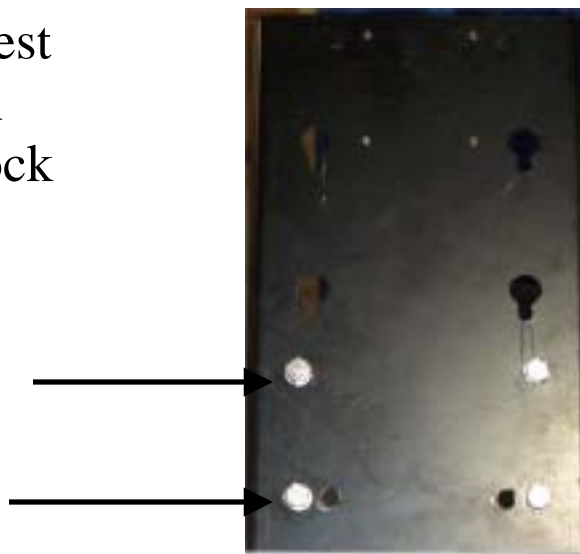

7. Its now time to mount the plastic backrest cover to the backrest mounting bracket. Use the four supplied Phillps head screws to complete this step.



8. Slide the headrest into position on the backrest mounting bracket. Remove the allen screw to install. Re-insert allen screw to lock headrest into channel.



9. Place the backrest onto the backrest mounting plate. Line up the four bolts into position with the backrest mounting plate. Once the bolts are lined up slide the backrest down into position. This will lock the backrest cushion into place.



10. Take the chair cushion and slide it into position then replace the two Allen screws into the bottom of the chair cushion.

