

The kit includes the following items.

QTY 1 DC-RP-26 Compressor head intake filter.


QTY 1 DC-RP-38.1 Particulate filter service kit (5-micron filter with clear bowl).

QTY 1 DC-RP-36 Parker Coalescing Filter (100-12-DX) B420



NOTE: The compressor must be off to change the filters.

Loosen the air filter inlet cap by turning the nut counterclockwise. Remove and discard the old filter by pulling it out. Replace the old filter (DC-RP-25) with the new one and secure the cap. Tighten the nut to secure the cap in place.

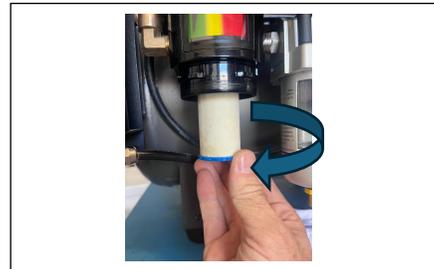


Push up on the coalescing filter housing and turn it clockwise to unseat it from the body. Unscrew the filter body by rotating the bottom cap counterclockwise. Remove the filter and replace the coalescing filter using the existing cap to secure it back in the housing.



Note the orientation of the top cap of the filter holder. Align the coalescing filter housing and gently turn it while pressing up to lock it back in place.



Slide the lock lever down to unlock the body housing and turn it in either direction to remove the lower casing. Gently pull the lower body down away from the upper housing.

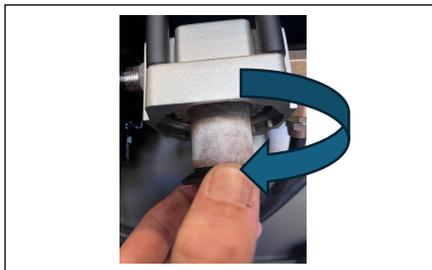

Unscrew the filter from the body by turning the plastic portion of the bottom of the filter counterclockwise. Note the orientation of the filter as shown above, before reinstallation of the new filter element and assembly. Discard the old filter and the old bowl, as the kit comes with a new filter and a new bowl with the O-ring pre-installed.



There are 4 positions the lever can sit in. Pull the lever down on the bowl housing back and align the notches of the housing, then turn the housing till the lever locks in one of the notches of the cap.