


## AIC 5855 Wireless Camera & AIC 5925B Docking Station Installation Instructions

- 1. Install the Imaging Software**
  - Download and install the imaging software application.  
(Download link: [HERE](#) )
- 2. Install the Camera Driver**
  - Before connecting the docking station via USB, install the camera driver:  
5925\_Driver.  
(Download link: [HERE](#) )
- 3. Restart Your Computer**
  - After installing the driver, restart your computer to complete the installation process.
- 4. Connect the USB Cable**
  - Plug the USB-A end of the cable into your computer.
  - Connect the “USB OUT” to the back of the docking station.




- 5. Verify Installation**
  - Open **Device Manager** on your computer.
  - Under **Imaging Devices**, confirm the camera appears as:  
"USB 2821 Video"

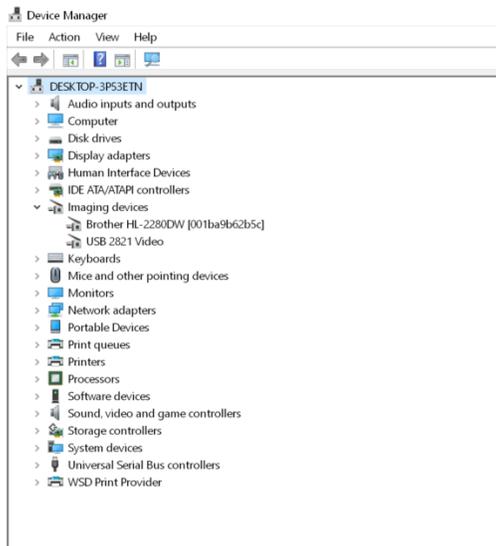

## Camera Control Functions

- 1. Power On/Off**
  - To **turn the camera ON**, press and hold the **Freeze/Save** button for 5 seconds.
  - To **turn the camera OFF**, press and hold the **Freeze/Save** button again for 5 seconds.
- 2. Freeze and Save Image**
  - Once a live image appears in the software or on an external display, **press the Freeze/Save button briefly** to freeze the image.
  - **Press the button again** to save the frozen image.
- 3. Full/Quad Screen Mode**
  - Press the **Full/Quad** button to switch to quad-view mode (4 images displayed in a 4-part grid).
  - In this mode, each time you press the **Freeze/Save** button, the camera captures an image and automatically moves to the next quadrant.
  - Press the **Full/Quad** button again to return to full-screen or single-image mode.
- 4. Scroll Through Saved Images**
  - Press the **Up** button to scroll **up** through saved images stored in the docking station.
  - Press the **Down** button to scroll **down** through the saved images.
- 5. Delete an Image**
  - To delete a saved image, press the **Delete** button.
  - Press the **Delete** button again to confirm and permanently remove the image.

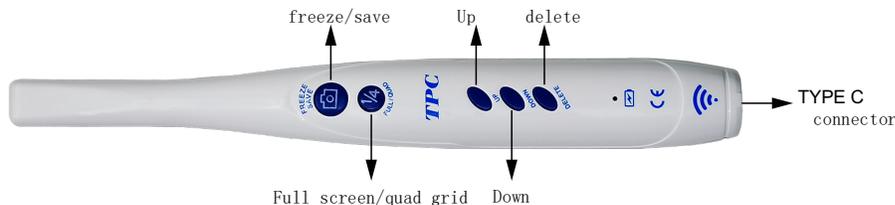

## Instructions for charging the camera.

- 1. Connect the USB-C cable:**
  - Plug one end into the **back of the camera**.
  - Plug the other end into the **USB 2.0 port** on the **docking station**.
- 2. LED Indicator Behavior:**
  - **Red LED** = Charging in progress.
  - **Green LED** = Charging complete.



## Docking Station Port Descriptions

- **USB OUT (Type B):** Sends data to a computer.  
✂ *Requires USB driver installation. 5925\_Driver. (Download link: [HERE](#))*
- **AV Out:**  
Outputs **composite video** to a display device (e.g., TV or monitor with AV input).
- **VGA Out:** Outputs **VGA video** to a compatible display device.
- **HDMI Out:** Outputs **HDMI video** to a display device (e.g., HDTV, monitor).
- **USB 2.0:** Used to **charge the AIC 5855 IO camera** via the USB-C port on the back of the camera.
- **DC IN (9V):** Powers the docking station using a **9V DC adapter**.
- **RF Antenna Port:** Connects a **wireless antenna** to the docking station for RF communication.
- **ON / OFF Switch:** Power the docking station **on or off**.

