

Advance 2500 Additional Notes!

To turn off the music that the adv 2500 plays when in use, please follow the steps below.

1. With the power OFF press and hold anywhere on the touch screen.
2. With your finger on the touch screen turn the power switch on the back of the unit to the ON position.
3. The Bleaching light will enter learning mode.
4. Follow the on screen instructions.
5. The screen will ask you where the 100% light intensity button is located. Press the corresponding button. Press where the 100% location would be all the way to the right of the intensity adjustment bar. Once you press the selection the screen will give you X & Y axes locations.
6. The next screen will ask you for the timer 20 min position. Press where the maximum position is on the timer location. This is on the Timer Adjustment Bar all the way to the right. Once you press the selection the screen will give you X & Y axes locations.
7. The screen will now ask you where the start / pause button is. Press the green start atop button to program this selection. Once you press the selection the screen will give you X & Y axes locations.
8. The screen will now ask you where the stop button is located. Press the RED stop button to program this location. Once you press the selection the screen will give you X & Y axes locations.
9. The screen will now ask you if you want the MUSIC ON or OFF. Press the corresponding button to turn the music on or off. Start / Pause will turn the music on. Stop will turn the music off.
10. The screen will now show intensity 100% adjusting. You may press the stop button to end this process and exit the learning mode. The screen will read exit learn mode bye!!!.
11. You have now programmed the buttons and turned the music on and off.