


L570 LED Operatory Light Quick Reference Installation Guide

ASSEMBLING THE HEAD TO THE ARM

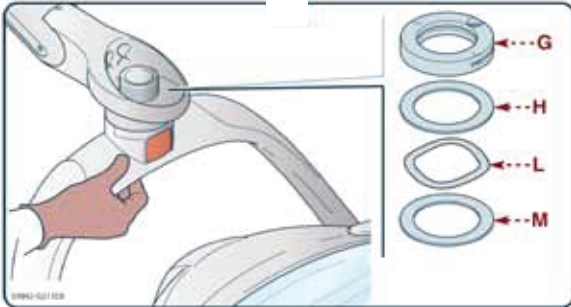


INFORMATION

Follow the assembly sequence of the ring nut and washer indicated in the picture.

Proceed according to the description given

- Support the head and insert the washers (H), (L), (M).
- Screw the ring nut (G).

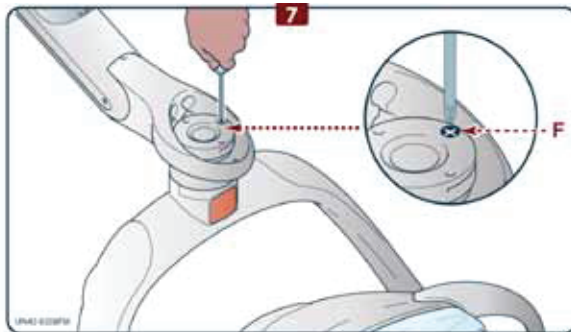

Tighten the safety screw (F) until the proper head rotation resistance is obtained.


Operation

Switch on / off / Adjustment

- To switch the lamp On and Off, toggle the switch to the LEFT to operate in sensor mode.
- Toggle to the RIGHT to operate in sensor mode. Note:


DON'T push and pull the switch. It only moves in a left and right direction.

Brightness adjustment of White light

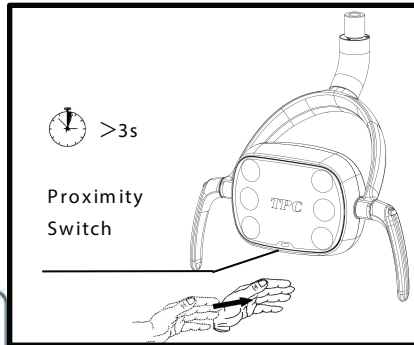

- ✓ The operating light is switched ON in the normal mode.
1. Hold your hand on the proximity switch for at least 3s.
 2. The operating light switches OFF once indicates it entered the setting mode and then increases the brightness setting to the maximum level step by step. Then the operating light reduces the brightness step by step.
 3. As soon as the desired brightness has been reached, pull your hand back from below the proximity switch. The current brightness setting is then saved for the normal mode. The light briefly flashes once again. This indicates that you have left the programming mode. The light switches to the normal mode.

Switching the operating light to the composite function

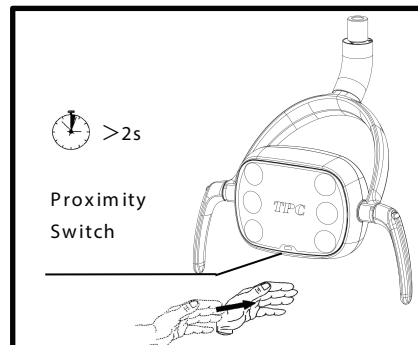

- The operating light is switched off.
1. Move your hand pass the sensor-controlled switch in 2s.
The operating light switches to normal mode with white light.
 2. Move your hand pass the sensor-controlled switch again after 2s,
The operating light switches to composite mode with yellow light.