


Main Chair PCB to foot control wire and pin layout

Pin #

1	○	Foot Control Trigger
2	○	Pin 1-4 Back Down
3	○	Pin 2-4 Back Up
4	○	Pin 1-3 Base UP
5	○	Pin 2-3 Base Down
6	○	Pin 4-6 Preset 1
7	○	Pin 1-7 Preset 2
		Pin 3-6 Zero Auto Return
		Pin 1-5 LP Last position backrest

Foot Control PCB

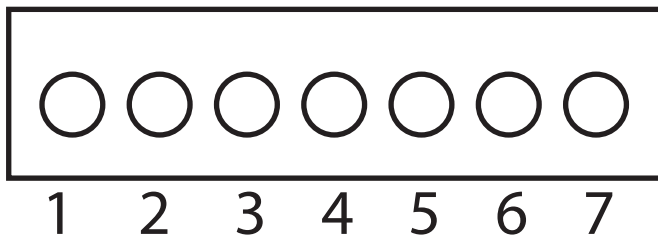

Wire Connections

- 1.Green
- 2.Yellow
- 3.Grey
- 4.Blue
- 5.White
6. Black
- 7.Red