

Mirage 2.0 Brake Caliper Removal Procedure

1. Open the brake access panel under the flex arm.



2. Remove the Air supply line from the brake valve.




3. Loosen the two screws that secure the brake caliper.



4. Use suction from a saliva ejector to retract the calipers.



5. You will not be able to remove the brake body if a caliper is extended.

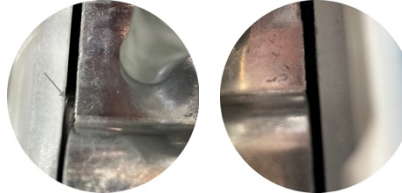

6. Use a utility knife to gently press against the side of the caliper. This will assist with the retraction process.



7. Once retracted, Remove the brake assembly.



8. Inspect for cracks, Remove debris, and lubricate the O-ring on the caliper pins.



9. Reverse steps to install new brake assembly.

