


2530 updated sensor Probe assembly and Main PCB

Important: When using the new sensor probe assembly, you must also replace the main PCB. Using the original PCB will eventually cause it to fail, as the R1 resistor now has a different rating.

Note: The R1 resistor's multiplier band is now **green** (see photos for reference).:

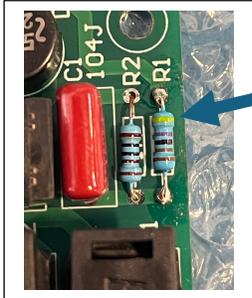

Note: The wire configuration has been updated for compatibility with the new sensor probes. All other existing wires should remain connected to the same ports on the new PCB.

The **black and red wires** that are soldered together have already been paired and should remain joined. Please refer to the attached diagram for proper wire alignment.

