

PC 2700 Portable Chair Operation Instructions.



PC 2700 Portable Chair Operation Instructions.

1. Place the chair on a flat surface as shown.



2. Loosen the safety knobs so the chair will travel in the height adjustment track.



3. Tilt the chair base on the rear legs. Place one foot on the front of the support frame and apply pressure to raise and lower the chair height to the desired level.



PC 2700 Portable Chair Operation Instructions.

4. Verify the adjustment studs are secure in the adjustment track prior to tightening the knobs. Failure to verify the studs are secure in the track may result in injury or damage to the product. Tighten the safety knobs that were loosened in step #2.



5. Place the headrest cushion in the desired position.



6. Secure the Headrest in place using the Velcro strap on the back of the backrest.



PC 2700 Portable Chair Operation Instructions.

7. Align the backrest adjustment stem into the backrest adjustment mechanism. Press down on the black lever to adjust the backrest to the desired height.



8. Install the optional light/cuspidor support post.




9. Secure the post in place with the included knob.




PC 2700 Portable Chair Operation Instructions.


10. Place the cuspidor support arm over the support post as shown.

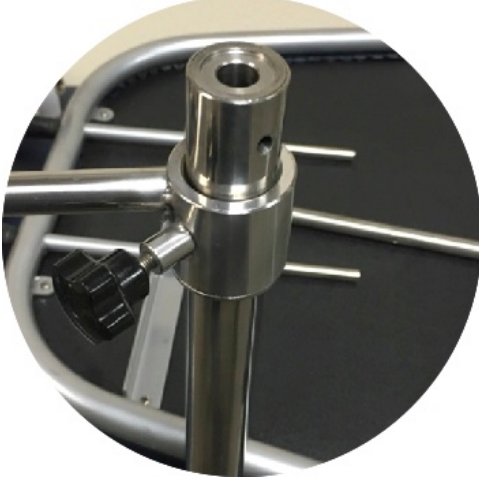

11. Place the stainless-steel tray support arm over the support post.

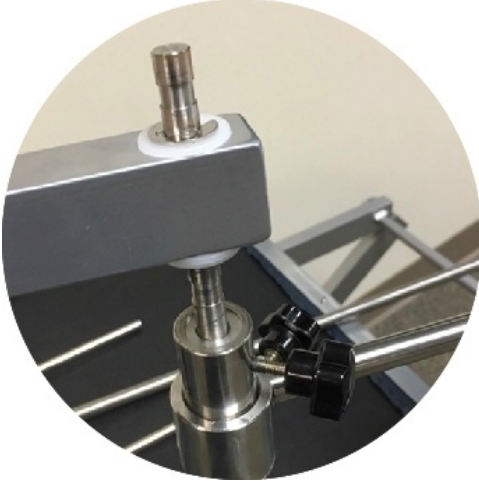

12. Place the LED light onto the support arm. Verify all the adjustment knobs are securely tightened. Failure to secure the adjustment knobs may result in damage to the equipment or injury to the operator/patient.

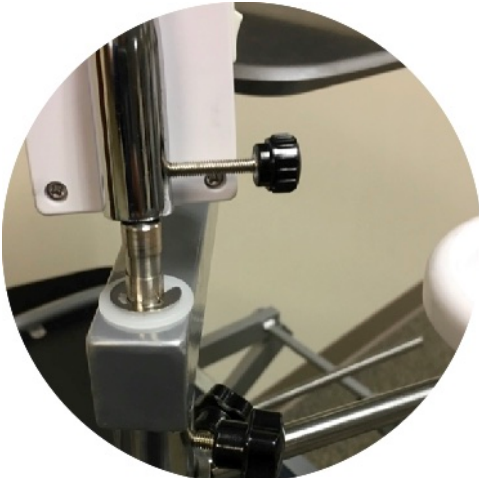

PC 2700 Portable Chair Operation Instructions.

Specifications:

Height adjustment of seat: 16½"–25 ½"

Overall Length of Chair: 54"

Overall Width of Chair: 18"

Backrest incline: 110 degrees – 175 degrees

Power of LED light: 12Vdc

Waste container: 1,000ml

Weight capacity: 250 LBS

PC 2700 Portable Chair Operation Instructions.

Safety Precautions

TPC accepts no liability for direct or consequential injury or damage resulting from improper use or incorrect assembly.

To avoid tipping hazards, never sit on the edge of the seat — always sit in the center of the seat.



Always use the chair on a hard flat surface.



The chair must be unoccupied when making height adjustments to the chair.



Before using the Chair, verify that the adjustment studs have been fully seated in the track. Verify the adjustment knobs are tightened.



Do not alter the chair assembly.



Do not attempt to repair or modify the structure of the chair frame.

PC 2700 Portable Chair Operation Instructions.

Exploded drawing of Portable Dental Chair Package PC2780

