

TPC

PC 2700 Portable Chair



Installation, Operation & User Manual

TPC Dental
851 S. Lawson St.
City of Industry, CA 91748
Phone: 626-810-4337
Fax: 626-810-4245
Web: www.tpcdental.com

Document Version: 2.0
Last Updated: October 2025
Part Number: PC-2700-MAN-001

Document Version: 2.0. Last Updated: October 2025 © TPC Dental. All rights reserved.



Assembly and Setup Instructions

Step 1: Initial Placement

Place the chair on a flat surface as shown.



Step 2: Loosen Safety Knobs

Loosen the safety knobs so the chair will travel in the height adjustment track.



TPC

Step 3: Adjust Chair Height

Tilt the chair base on the rear legs. Place one foot on the front of the support frame and apply pressure to raise and lower the chair height to the desired level.



Step 4: Secure the Adjustment

⚠ IMPORTANT: Verify the adjustment studs are secure in the adjustment track prior to tightening the knobs. Failure to verify the studs are secure in the track may result in injury or damage to the product.

Tighten the safety knobs that were loosened in Step 2.



TPC

Step 5: Install Headrest Cushion

Place the headrest cushion in the desired position.



Step 6: Secure the Headrest

Secure the headrest in place using the Velcro strap on the back of the backrest.



TPC

Step 7: Adjust Backrest Height

Align the backrest adjustment stem into the backrest adjustment mechanism. Press down on the black lever to adjust the backrest to the desired height.



Step 8: Install Optional Light/Cuspidor Support Post

Install the optional light/cuspidor support post.



TPC

Step 9: Secure the Support Post

Secure the post in place with the included knob.



Step 10: Install Cuspidor Support Arm

Place the cuspidor support arm over the support post as shown.



Step 11: Install Stainless Steel Tray Support Arm

Place the stainless-steel tray support arm over the support post.



Step 12: Install LED Light

Place the LED light onto the support arm.



⚠ IMPORTANT: Verify all the adjustment knobs are securely tightened. Failure to secure the adjustment knobs may result in damage to the equipment or injury to the operator/patient.



Specifications

Specification	Measurement
Height Adjustment of Seat	16½" – 25½"
Overall Length of Chair	54"
Overall Width of Chair	18"
Backrest Incline	110 degrees – 175 degrees
Power of LED Light	12V DC
Waste Container Capacity	1,000 ml
Weight Capacity	250 lbs

Safety Precautions

⚠ IMPORTANT SAFETY INFORMATION

TPC accepts no liability for direct or consequential injury or damage resulting from improper use or incorrect assembly.

Safety Guidelines:

- **To avoid tipping hazards, never sit on the edge of the seat** — always sit in the center of the seat
 - **Always use the chair on a hard, flat surface**
 - **The chair must be unoccupied when making height adjustments to the chair**
 - **Before using the chair, verify that the adjustment studs have been fully seated in the track.** Verify the adjustment knobs are tightened.
 - **Do not alter the chair assembly**
 - **Do not attempt to repair or modify the structure of the chair frame**
-



Maintenance and Care

Regular Inspection

- Check all adjustment knobs for tightness before each use
- Inspect adjustment studs and tracks for wear or damage
- Verify Velcro straps are secure and functioning properly
- Check LED light connections and functionality

Cleaning

- Clean all surfaces with appropriate disinfectant
- Wipe down stainless steel components regularly
- Empty and clean waste container as needed
- Clean headrest cushion according to infection control protocols

Troubleshooting

Problem	Possible Cause	Solution
Chair height won't adjust	Safety knobs are tightened Adjustment studs not in track	Loosen safety knobs before adjusting Verify studs are properly seated in track
Chair feels unstable	Adjustment knobs not tightened Uneven surface	Tighten all adjustment knobs securely Move chair to flat, level surface
Backrest won't adjust	Lever not fully depressed Adjustment mechanism jammed	Press down firmly on black lever Contact TPC service for assistance
LED light not working	Power not connected Bulb burned out	Check 12V DC power connection Replace LED bulb
Headrest won't stay in place	Velcro strap loose	Secure Velcro strap tightly



Warranty Information

All TPC products sold are guaranteed to be free from defects in workmanship and materials for one year from the date of purchase, unless otherwise stated. TPC will repair or replace any defective part at no charge. TPC will not be responsible for labor charges or shipping charges to/from the TPC facility. This guarantee does not cover normal wear or stains on the surface finish. The guarantee does not cover damage resulting from improper installation, misuse, or accidents incurred in shipping and handling.

Contact Information

TPC Dental

851 S. Lawson St.

City of Industry, CA 91748

Phone: 626-810-4337

Fax: 626-810-4245

Toll-Free: 800-560-8222

Web: www.tpcdental.com

Email: service@tpcdental.com

Important Notes

- **Always read and follow all safety precautions before using the chair**
 - **Keep this manual for future reference**
 - **Contact TPC for replacement parts or service questions**
 - **Only use genuine TPC replacement parts**
-

End of Manual