



SS-5109 Saddle Stool Assembly instructions

This stool requires a limited amount of assembly. Please follow the assembly instructions below.

1. Insert Caster wheels (1) into support base (2).
2. Insert gas cylinder (3) into support base (2).
3. Attach seat cushion (4) onto gas cylinder (3).

Stool Function Adjustments

Seat Height

The height of the seat can be adjusted by pulling up on height adjustment lever (5). Applying and removing pressure on the seat will raise and lower the height. Release adjustment lever (5) once desired height has been reached.

Seat / Back tilt

The seat will tilt -5 degrees forward and 6 degrees backwards. The backrest has 10 degrees of forward tilt and 14 degrees of backward tilt. By pushing lever (6) downward to the "free float" position, the backrest and seat tilt will move freely. Once in the desired position pull adjustment lever (6) up to lock in place.

