

H7040 Fiber Optic Light Source installation Procedure

H7000 5 Hole System



H7660 6 Pin System



1. Determine which position you would like to install the power optic light source tubing.
2. Connect the 4 handpiece tubing's connections to the proper ports on your delivery unit's control block.
3. Place the light source control box in the desired location close to the delivery unit head.
4. Connect the input power to the IN ports on the control board. The use of low voltage wire will be required. Connect the provided AC power adaptor to the other end of the low voltage wire.
5. Connect the two wires from the hand piece tubing to your control box terminals A or B. there are two terminals for each letter.
6. Connect the signal air from the drive air line to the signal air port on the control box. The signal air must connect to the same letter port as the electrical connection that was used.
7. Use the control adjustment on the control box to adjust the light intensity.

