



## TPC Unit Mounted X-Ray Viewer Bracket Installation




1. Find Desired location to mount your x-ray viewer bracket block. If bracket is being installed on unit's other then TPC you may need to drill holes in the delivery unit arm.



2. After holes are drilled or located take the mounting bolt out of the bracket block.



3. Place bolt through the hole in the unit arm.

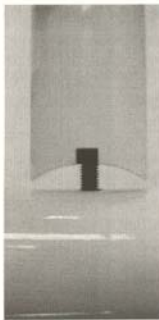

4. Take the mounting block and begin to screw the mount block bolt into the block. Make sure you completely tighten the bolt or damage may occur.



5. Place x-ray mounting bracket in channel of X-Ray mounting block till it stops. The image shown is only 25% down. Tighten the three setscrews on the backside of the mounting block.



6. Now mount the x-ray viewer bracket to the x-ray viewer as shown to the left. Place one end of the bracket in the channel. Then compress the two plastic parts together to lock the bracket down into the viewer.



7. DC Power in.  
Note: If there is low voltage wire ran through your delivery unit arm you may install transformer in post mount unit box. Otherwise plug transformer into wall outlet, then connect the other end to the X-Ray viewer.

