


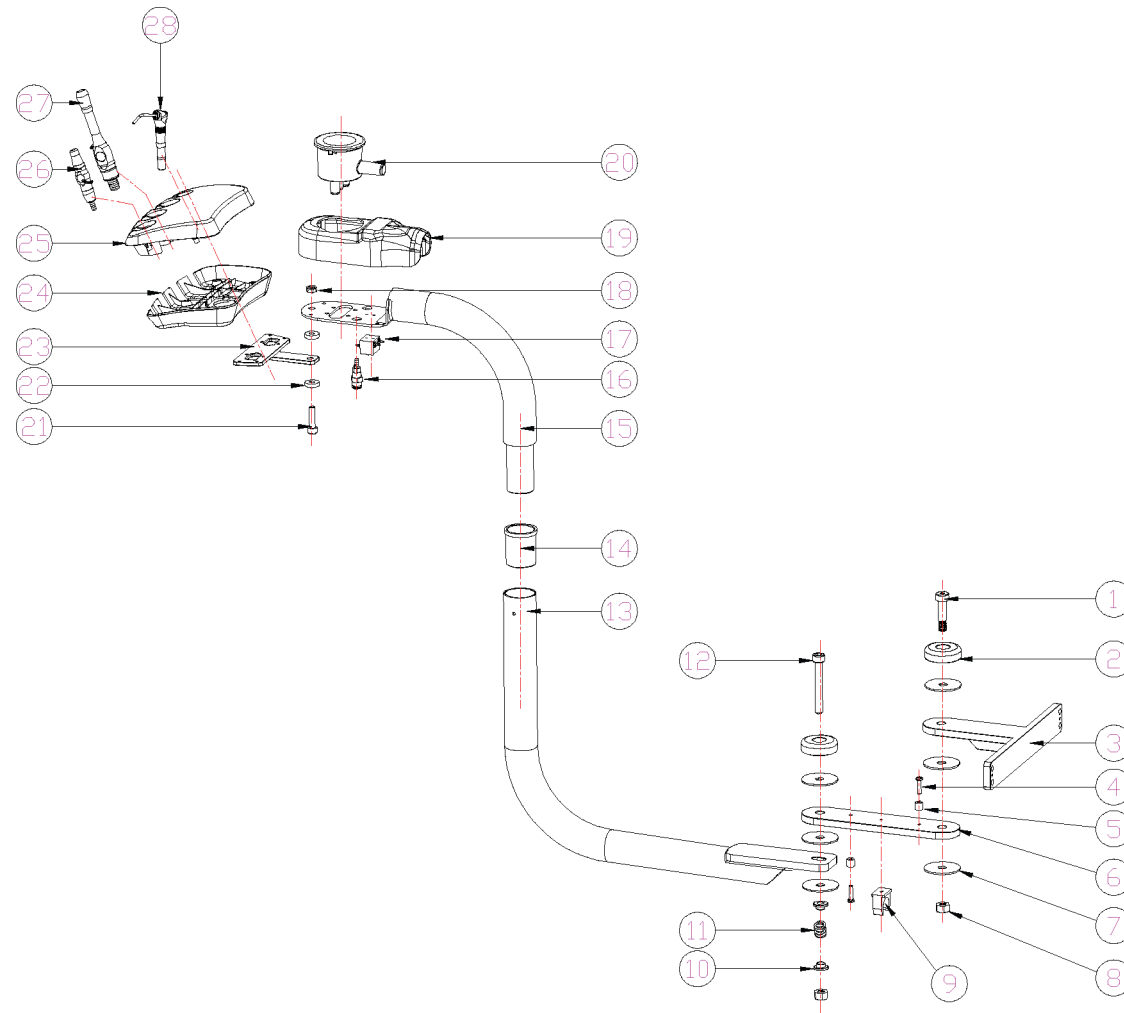

# Mirage 2.0 3550 Rear Vacuum Parts List

3550-2.0



TPC 851 S. Lawson St. City of Industry, CA 91748  
Phone 626-810-4337 Fax 626-810-4245



Mirage 2.0 3550 Rear Vacuum Parts List

Unit Head & Flex Arm

No.	Description	Part Number	No.	Description	Part Number
1	Allen Bolt	90001	26	Saliva Ejectors	P2340
2	Bolt Bezel Ring	90002	27	HVE Valve	P2335
3	Rear Bracket	90003	28	3 - Way Syringe	TPC3WAY-PR
4	Stop Bolt	90004			
5	Nylon Stand Off	90005			
6	Bracket Extension	90006			
7	Nylon Washer QTY 3	90007			
8	Nut	90008			
9	Tubing Holder Clip	90009			
10	Spring Base	90010			
11	Tension Spring	90011			
12	Bolt	90012			
13	Lower Vacuum Arm	90013			
14	Nylon Bushing	90014			
15	Upper Vacuum Arm	90015			
16	Quick Disconnect	90016			
17	Syringe Block	90017			
18	Nut	90018			
19	Shroud Cover	90019			
20	Vacuum Canister	90020			
21	Bolt	90021			
22	Nylon Spacer	90022			
23	Lower HP Frame Bracket	90023			
24	Rear Vacuum Lower Cover	90024			
25	Rear Vacuum Upper Cover	90025			