

## MC 3600 CV-SL Hand Piece Holder Replacement / Adjustment.


1. Disconnect the Machine from the power outlet.
2. Using a 2mm Allen wrench loosen the two set screws on the bottom of the holder.




3. Once the screws are loose, rotate the holder so that you have access to the back of the holder's cap. Remove the two Phillips screws and remove the back cap.




4. Remove the holder from the bar. Loosen the 2mm Allen set screw that holds the switch in.

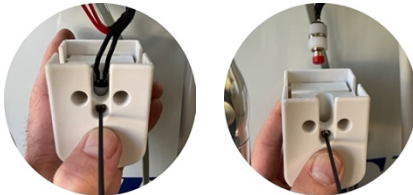


5. Seat the SE valve in the holder. Install the micro switch into the holder. Press the switch in so that the plunger on the holder activates the switch. Secure the micro switch by tightening the set screw.

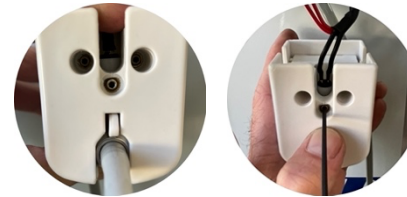

6. Place the holder back on the holder bar. Replace the back cap cover of the handpiece holder shell.

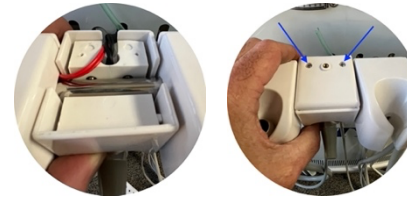

7. Align the HP holder to the desired position. Using the 2mm Allen wrench, Secure the HP holder set screws to the holder bar.

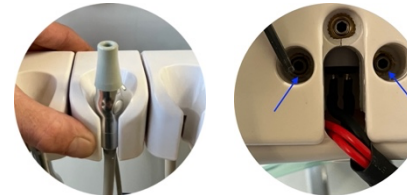

8. Connect the main power source and test.