


Mirage Delivery System Parts List




Mirage Sector Head Assembly

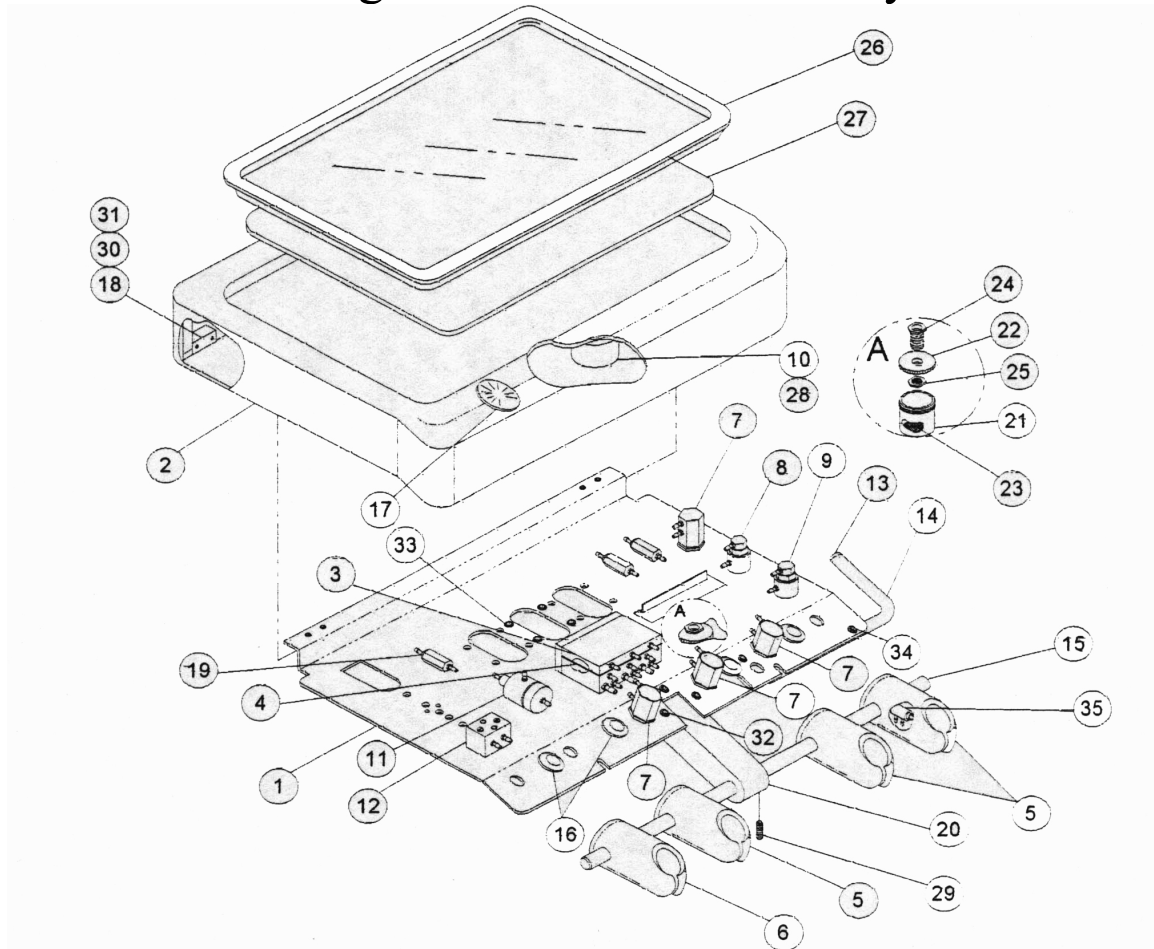


Item	Part Number	Description
1	52001	Plate Base
2	52002	Cover- Control Head
3	52003	Hand piece Control Block
4	52004	Tri-Block Diaphragm
5	52005	Holder Asepsis (Cover)
	52005-C	Asepsis Holder Complete
6	52006	Holder Syringe (Cover)
	52006-C	Syringe Holder Complete
7	52007	Valve-Flow Control Needle
8	52008	Push Button Momentary
9	52009	On/Off Toggle 3-Way
10	52010	Weight, Cover
11	52011	Water Relay
12	52012	Syringe Block Assembly
13	52013	Break Push Button
14	52014	Handle, Right
15	52015	Bar Holder
16	52016	Grommet
17	52017	Gauge-Pressure
18	52018	Hinge
19	52019	Check Valve
20	52020	Rod Holder Support
21	52021	Oil Jar
22	52022	Cap-Oil Jar
23	52023	Oil Pad
24	52024	Hex Screw-Oil Jar
25	52025	Hex Nut Oil Jar
26	52026	Stainless Steel Tray
27	52027	Non Slip Pad
28	52028	Screw, Button Head
29	52029	Screw, Set Socket
30	52030	Screw, Button Head
31	52031	Screw, Button Head
32	52032	Screw, Button Head
33	52033	Screw, Button Head
34	52034	Screw, Button Head
35	52035	N.C. Valve
36	52036	Hand Piece Tubing
37	52038	Syringe Tubing

Mirage Post Mount Utility Box

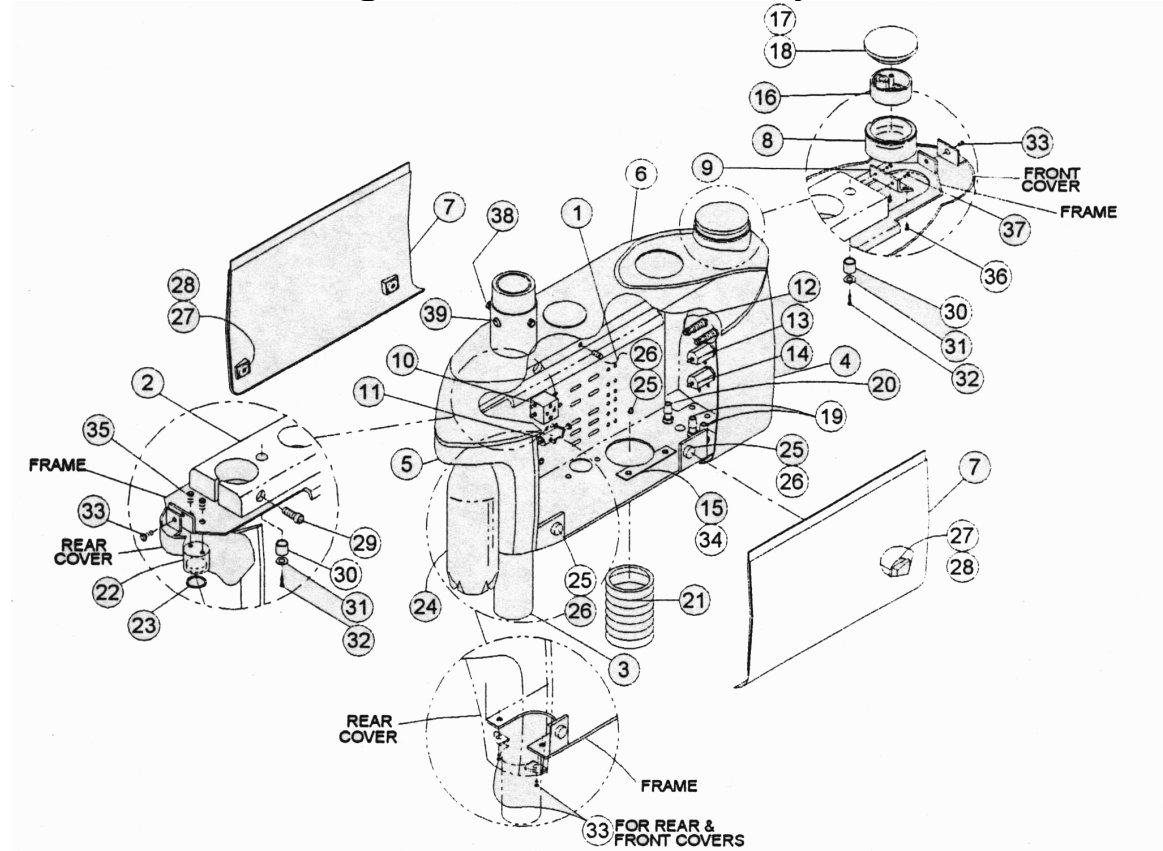

Item	Part Number	Description
1	53001	Frame, PMU
2	53002	Support Plate –Clamp
3	53003	Support Post
4	53004	Front Cover
5	53005	Rear Cover
6	53006	Top Cover
7	53007	Side Cover
8	53008	Solids Collector body
9	53009	Bracket, Solids Collector
10	53010	Syringe Block Assembly
11	53011	Water Bottle Air Regulator
12	53012	Q.D. Outlet W/Shutoff
13	52007	Flow Control Needle Valve
14	53014	Toggle, Selector Valve
15	53015	Umbilical Retainer Bracket
16	53016	Trap Solids Collector
17	53017	Cap, Solids Collector
18	53018	O-Ring Solids Collector
19	53019	HVE Connector
20	53020	SE Connector
21	53021	Umbilical (Per Foot
22	53022	Bottle Cap Assembly
23	53023	Gasket Water Bottle
24	53024	Water Bottle
25	53025	Magnet
26	53026	Screw Magnet
27	53027	Plate Magnet
28	53028	Screw Plate Magnet
29	53029	Screw, Clamp Mount Allen
30	53030	Spacer, Top Cover
31	53031	Washer, Top Cover
32	53032	Screw, Top Cover
33	53033	Screw, Rear/Front Cover
34	53034	Screw, Bracket – Umbilical
35	53035	Screw, Bottle Cap Assembly
36	53036	Screw, Solids Collector
37	53037	Screw, Bracket-Solids Collector
38	53038	Screw, Set Socket, Post
39	53039	Screw, Pan Head, Post
40	53040	Light Post Not Shown
41	53041	4 Position Suction Holder

Mirage Flex Arm Assembly

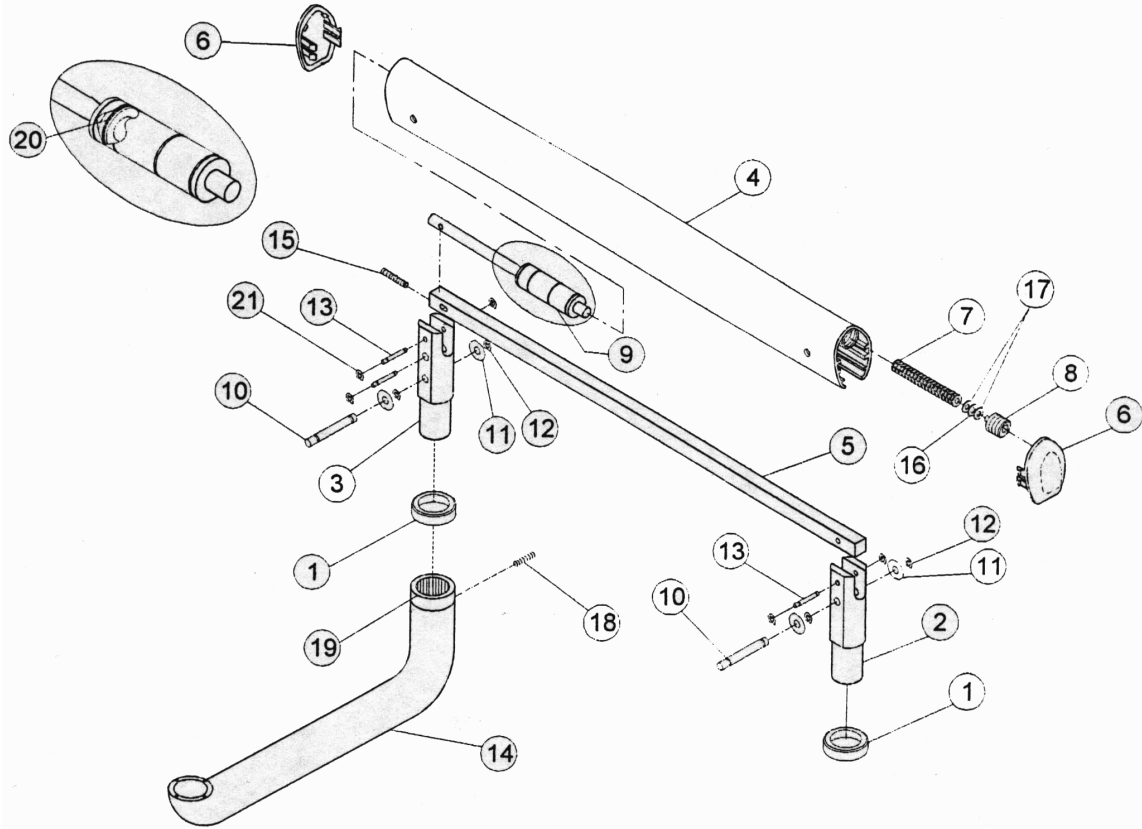




Mirage 2015 / 2000 parts list


<u>Item</u>	<u>Part Number</u>	<u>Description</u>	<u>Price\$</u>
1	54001	Bezel	
2	54002	Elevation Knuckle	
3	54003	Control Knuckle	
4	54004	Cover Flex Arm	
5	54005	Stabilizer Bar	
6	54006	End Cap, Flex Arm	
7	54007	Spring Compression	
8	54008	Tensioning Screw Cap	
9	54009	Air Break	
10	54010	Long Pin, Stabilizer	
11	54011	Washer (PKG 10)	
12	54012	Retainer (PKG 10)	
13	54013	Short Pin	
14	54014	Control Arm	
15	54015	Leveling Screw	
16	54016	Washer (PKG 10)	
17	54017	Washer (PGK 10)	
18	54018	Set Screw	
19	54019	Bearing	
20	54020	O-Ring Air Break	
21	54021	Retainer, Small (PKG 10)	

Mirage Cupidor – Post Assembly

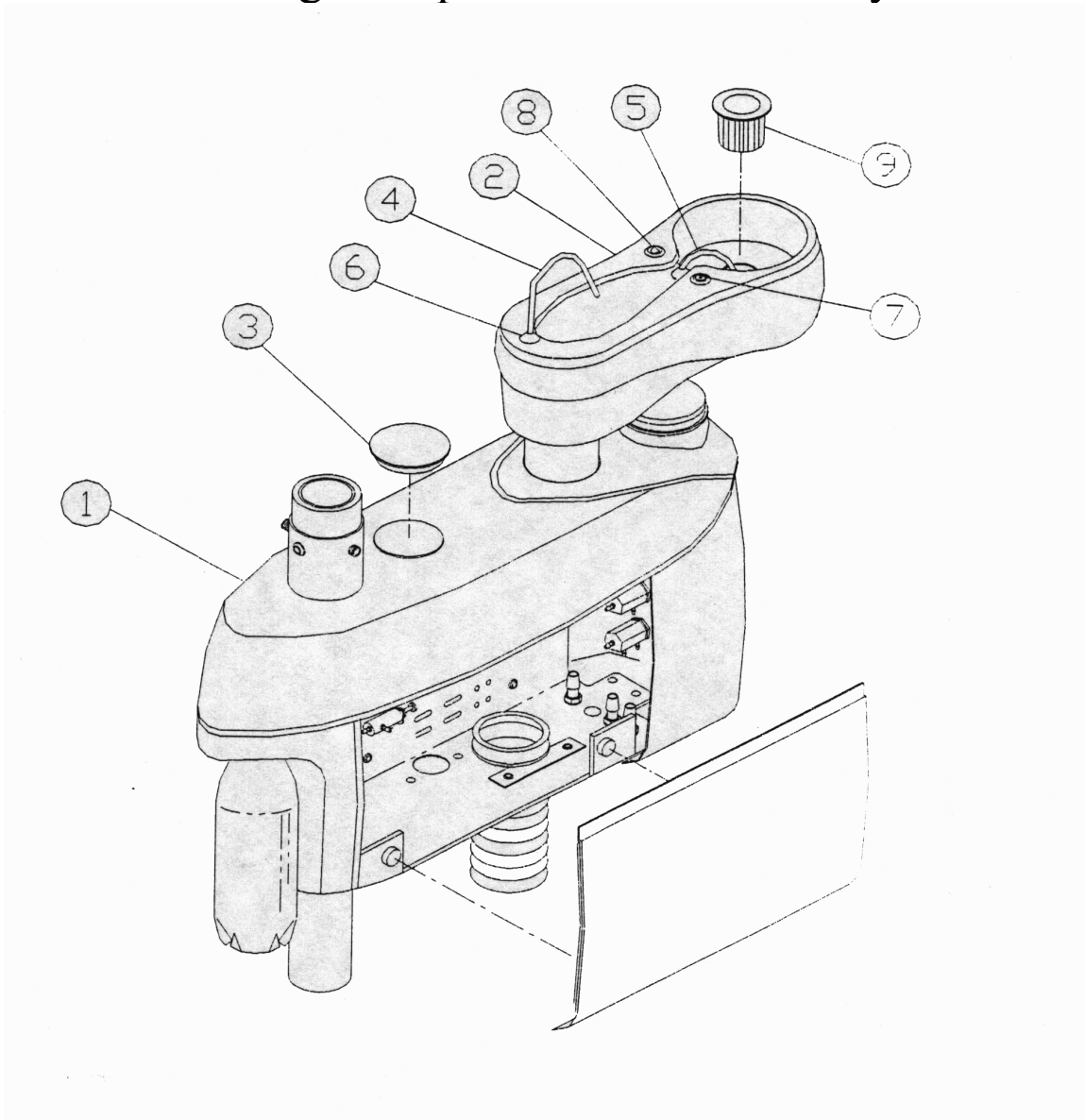


Item	Part Number	Description
1	53000	Post Mount Box Sub-Assembly
2	53050	Cupidor Sub-Assembly
3	53051	Cap, Top Cover
4	53052	Cup Fill Spout
5	53053	Bowl Rinse Spout
6	53054	Bezel, Spout
7	53055	Cup Fill Boot
8	53056	Bowl Rinse Boot
9	53057	Bowl Strainer
10	53058	Cupidor Cove Cap

Mirage Cuspidor Assembly

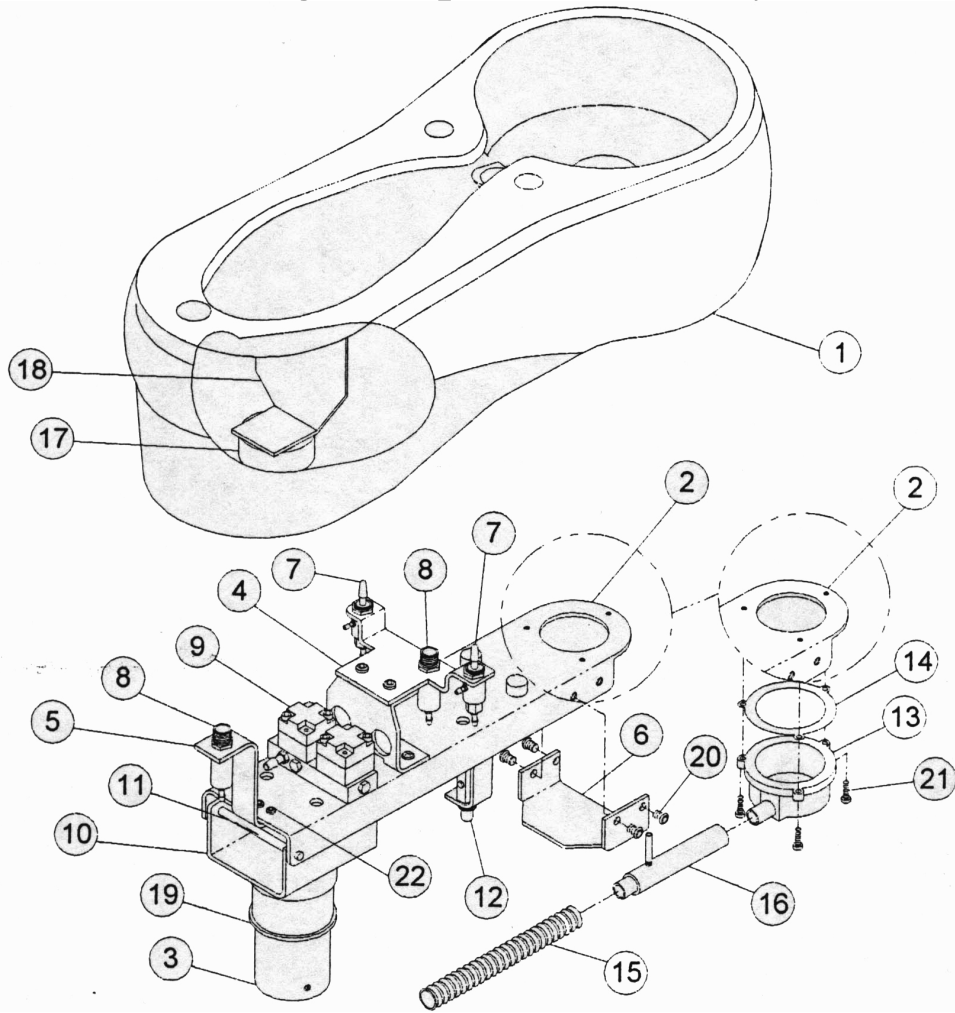

Item	Part Number	Description
1	53071	Cuspidor
2	53072	Chassis
3	53073	Pivot Tube
4	53074	Valve Bracket
5	53075	Bracket, Socket Cup Fill
6	53076	Bracket, Bowl- Socket
7	53077	Control Valve
	53077-C	Button Cap cover
8	53078	Socket Spout
9	53079	Block
10	53080	Bracket Pivot Mount
11	53081	Pin
12	52007	Needle Valve
13	53083	Socket Bowl
14	53084	Gasket
15	53085	Drain Line
16	53086	Vent Tube Assembly
17	53087	Magnet
18	53088	Bracket Magnet
19	53089	Washer
20	53090	Screw, Bracket Mount Allen
21	53091	Screw, Socket Mount Philips
22	53092	Screw, Bracket Cup Fill Allen

Mirage Utility Box Assembly

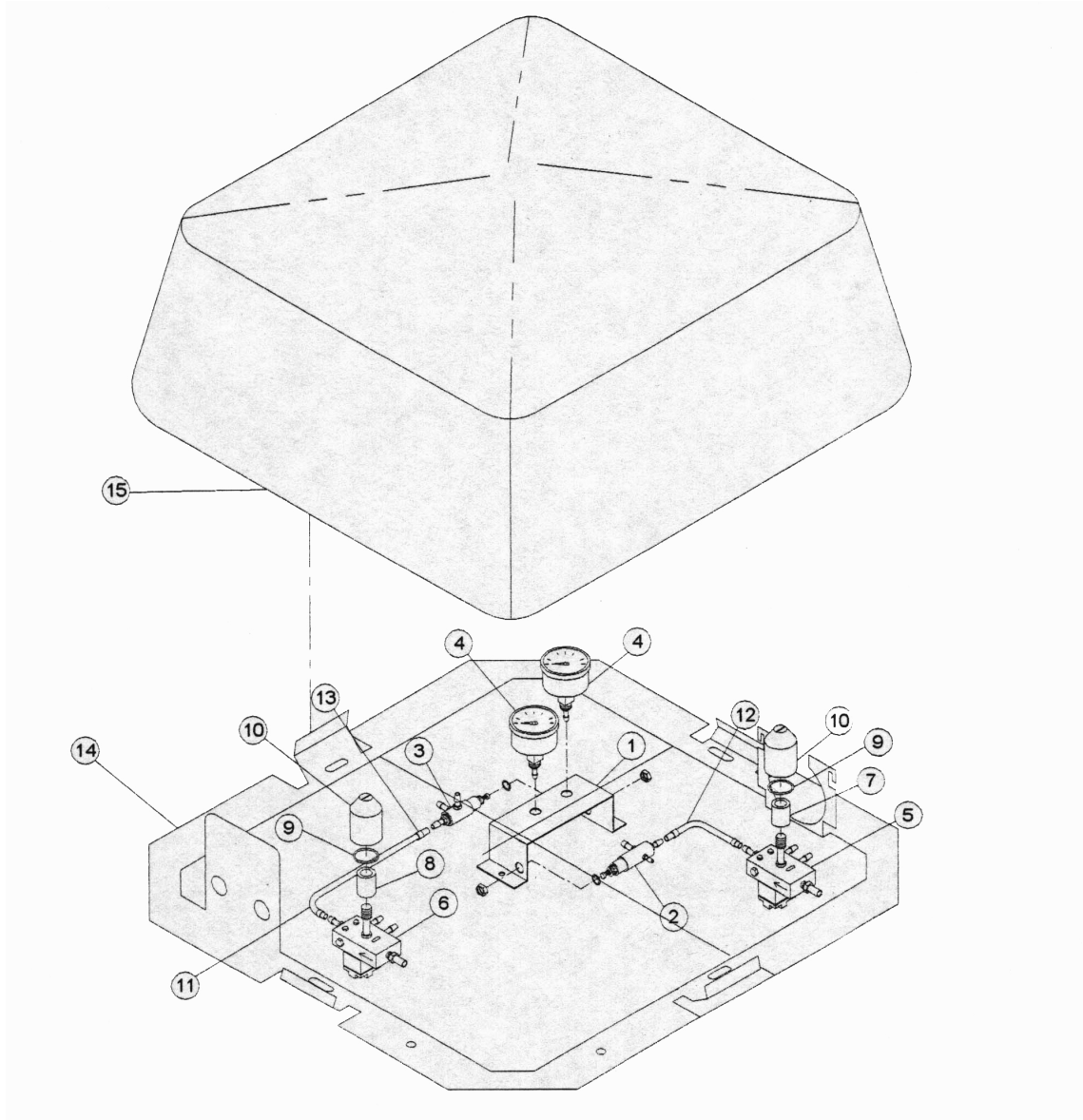




Mirage 2015 / 2000 parts list

Item	Part Number	Description
1	53101	Bracket
2	53102	Air Regulator
3	53103	Water Regulator
4	53104	Pressure Gauge
5	53105	Air Pilot Valve
6	53106	Water Pilot Valve
7	53107	Air Filter
8	53108	Water Filter
9	53109	O-ring
10	53110	Dome Nut
11	53111	¼" Water Line
12	53112	¼" Air line
13	53113	Sleeve Clamp (PKG 10)
14	53114	Frame, Utility Box
15	53115	Cover, Utility Box
16	53116	Foot Control W/SW
17	53117	Foot Control W/O SW
18	53118	F.C. Wet/Dry Toggle SW

Item	Part Number	Description
1	53071	Cuspidor
2	53072	Chassis
3	53073	Pivot Tube
4	53074	Valve Bracket
5	53075	Bracket, Socket Cup Fill
6	53076	Bracket, Bowl- Socket
7	53077	Control Valve
	53077-C	Button Cap cover
8	53078	Socket Spout
9	53079	Block
10	53080	Bracket Pivot Mount
11	53081	Pin
12	52007	Needle Valve
13	53083	Socket Bowl
14	53084	Gasket
15	53085	Drain Line
16	53086	Vent Tube Assembly
17	53087	Magnet
18	53088	Bracket Magnet
19	53089	Washer
20	53090	Screw, Bracket Mount Allen
21	53091	Screw, Socket Mount Philips
22	53092	Screw, Bracket Cup Fill Allen

Item	Part Number	Description	Price\$
1	53071	Cuspidor	\$100.00
2	53072	Chassis	\$50.00
3	53073	Pivot Tube	\$10.00
4	53074	Valve Bracket	\$5.00
5	53075	Bracket, Socket Cup Fill	\$2.00
6	53076	Bracket, Bowl- Socket	SEE NO.4
7	53077	Control Valve	\$19.00
	53077-C	Button Cap cover	\$1.00
8	53078	Socket Spout	\$12.00
9	53079	Block	\$33.00
10	53080	Bracket Pivot Mount	\$10.00
11	53081	Pin	\$1.00
12	52007	Needle Valve	\$16.00
13	53083	Socket Bowl	\$5.00
14	53084	Gasket	\$10.00
15	53085	Drain Line	\$1.98 Per Ft
16	53086	Vent Tube Assembly	\$4.00
17	53087	Magnet	\$20.00
18	53088	Bracket Magnet	\$5.00
19	53089	Washer	\$1.00
20	53090	Screw, Bracket Mount Allen	\$.40 Each
21	53091	Screw, Socket Mount Philips	\$.40 Each
22	53092	Screw, Bracket Cup Fill Allen	\$.40 Each

Assistants Arm

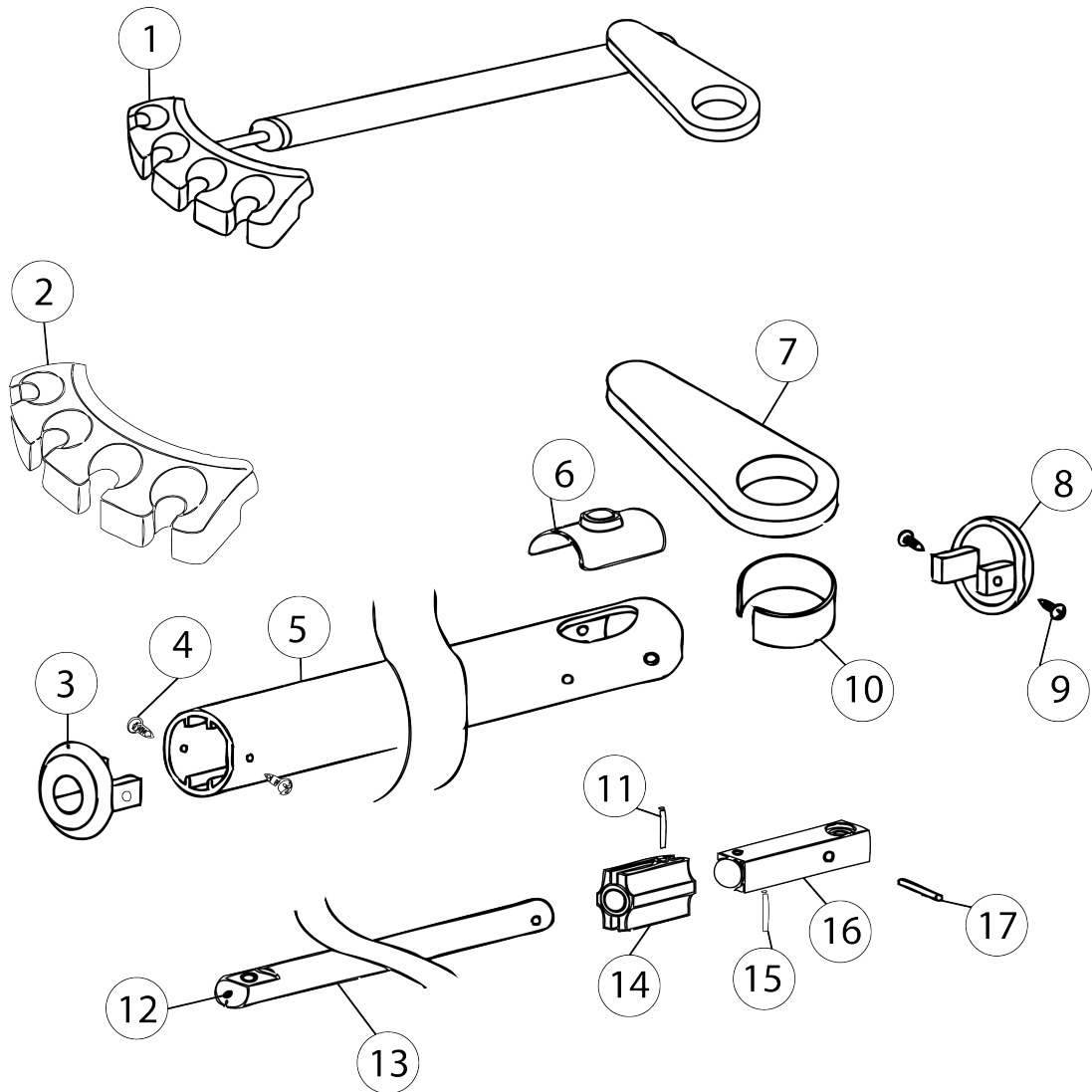



No.	Description	Part Number
1	Assistance Arm Complete	53041-C
2	4 Position Holder	53041
3	Arm Front Cover	53041-EC
4	Screw	53041-04
5	Arm Housing	53041-05
6	Trim Cover	53041-B-PC
7	Cast support	53041-B
8	Arm Back Cover	53041-08
9	Screw	53041-09
10	Bushing	53041-10
11	Pin	53041-11
12	Set Screw	53041-B-SS
13	Telescoping arm	53041-13
14	Block	53041-14
15	Leveling Screw	53041-BAB-LS
16	Break Away Block	53041-BAB
17	Pin	53041-17