




MC3600 Parts List

No.	Description	Part Number
1	Cabinet	OMC2375-RP1-1
2	Top cover Plastic	OMC2375-RP1-2
3	Pressure Regulator	OMC2375-RP1-3
4	Bracket	OMC2375-RP1-4
5	Transformer	OMC2375-RP1-5
6	Fan Cover	OMC2375-RP1-6
7	110V Fan	OMC2375-RP1-7
8	Tank	OMC2375-RP1-8
9	Solids Collector	OMC2375-RP1-9
10	Collector Bracket	OMC2375-RP1-10
11	Screw	OMC2375-RP1-11
12	Screw	OMC2375-RP1-12
13	Suction HVE Holder	OMC2375-RP1-13
14	Automatic HP Holder	OMC2375-RP1-14
15	Syringe Holder	OMC2375-RP1-15
16	Holder Bar	OMC2375-RP1-16
17	Screw	OMC2375-RP1-17
18	Power Terminal Block	OMC2375-RP1-18
19	Block Bracket	OMC2375-RP1-19
20	Fan Cover	OMC2375-RP1-20
21	Cap Bracket Screw	OMC2375-RP1-21
22	Cap Bracket	OMC2375-RP1-22
23	Compressor Head HP	OMC2375-RP1-23
24	Suction Motor	OMC2375-RP1-24
25	110V Inlet Port	OMC2375-RP1-25
26	Pressure Gauge	OMC2375-RP1-26
27	ON / Off Switch	OMC2375-RP1-27

No.	Description	Part Number
28	Power Switch Plate	OMC2375-RP1-28
29	Screw	OMC2375-RP1-29
30	Pressure ON / Off	OMC2375-RP1-30
31	Case	OMC2375-RP1-31
32	Gromit	OMC2375-RP1-32
33	Power Terminal Block	OMC2375-RP1-33
34	Screw	OMC2375-RP1-34
35	110V Outlet	OMC2375-RP1-35
36	110V fan	OMC2375-RP1-36
37	Pressure Switch	OMC2375-RP1-37


## MC3600 Parts List

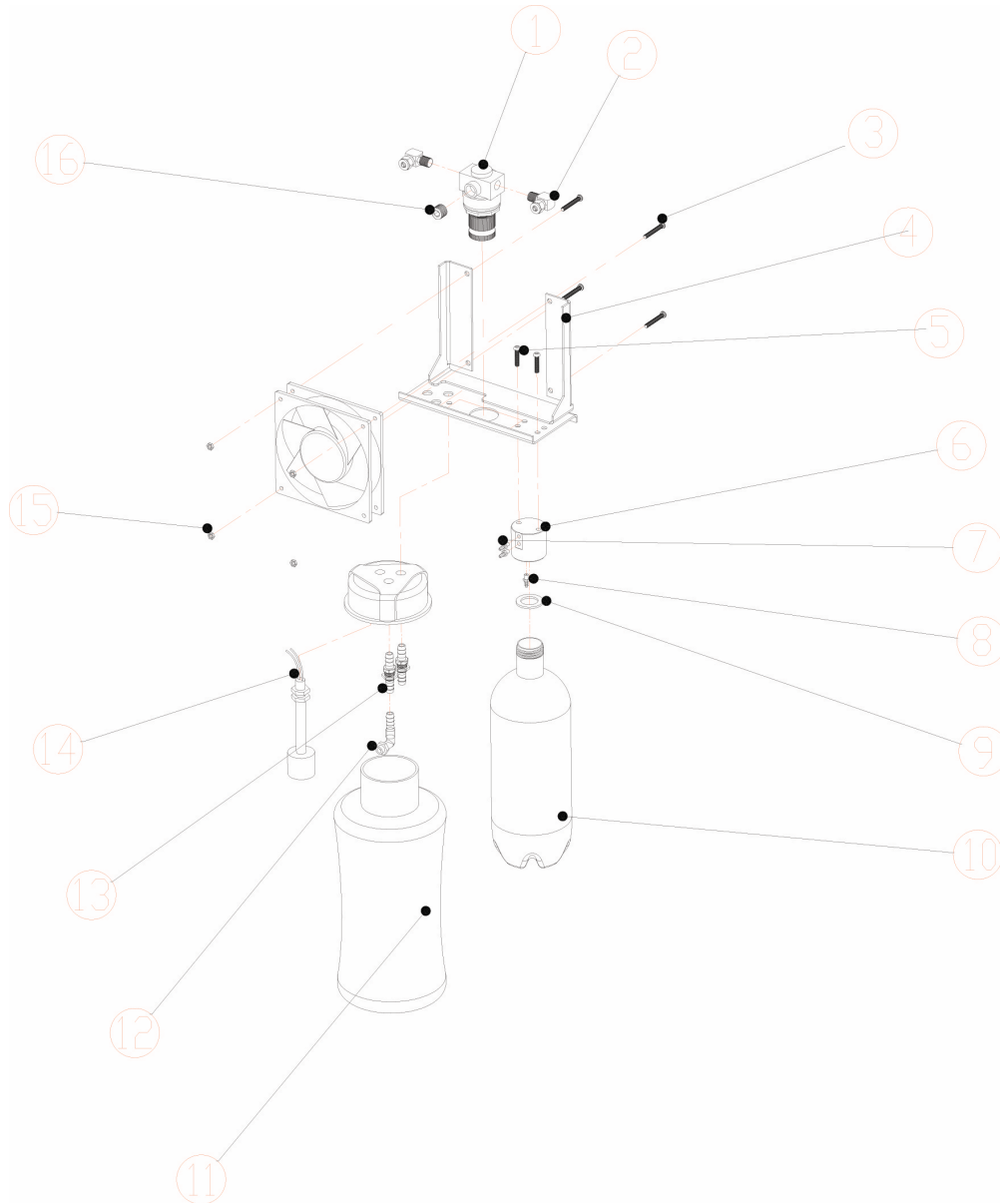


No.	Description	Part Number
1	Pressure Regulator	OMC2375-RP2-1
2	90-degree elbow	OMC2375-RP2-2
3	Screw	OMC2375-RP2-3
4	Bracket	OMC2375-RP2-4
5	Screw	OMC2375-RP2-5
6	Bottle Cap	OMC2375-RP2-6
7	Barb	OMC2375-RP2-7
8	Barb	OMC2375-RP2-8
9	Gasket	OMC2375-RP2-9
10	Bottle	OMC2375-RP2-10
11	Waste Container	OMC2375-RP2-11
12	90-degree spout	OMC2375-RP2-12
13	Straight Barb	OMC2375-RP2-13
14	Fill Sensor Probe	OMC2375-RP2-14
15	Nut	OMC2375-RP2-15



MC3600 Parts List

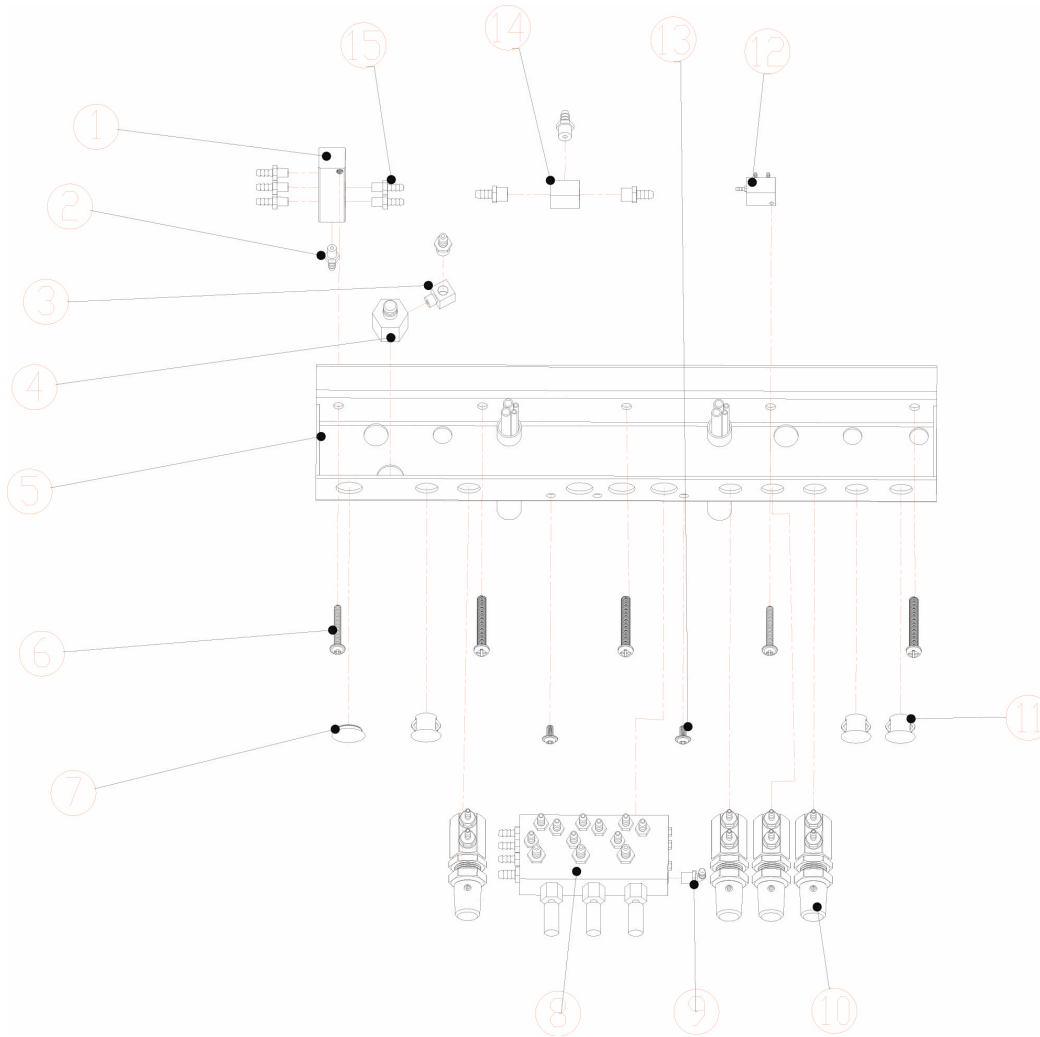

No.	Description	Part Number
1	6 way block	OMC2375-RP3-1
2	Barb	OMC2375-RP3-2
3	90 Degree Metal BK	OMC2375-RP3-3
4	90 Degree Body Bk	OMC2375-RP3-4
5	HPO Block Frame	OMC2375-RP3-5
6	Bottle Cap	OMC2375-RP3-6
7	Screw	OMC2375-RP3-7
8	Main Tri Block	OMC2375-RP3-8
9	90 Degree Barb	OMC2375-RP3-9
10	Needle Valve	OMC2375-RP3-10
11	Plastic Cover Cap	OMC2375-RP3-11
12	Water Relay	OMC2375-RP3-12
13	Screw	OMC2375-RP3-13
14	Three way TEE body	OMC2375-RP3-14
15	Barb	OMC2375-RP3-15

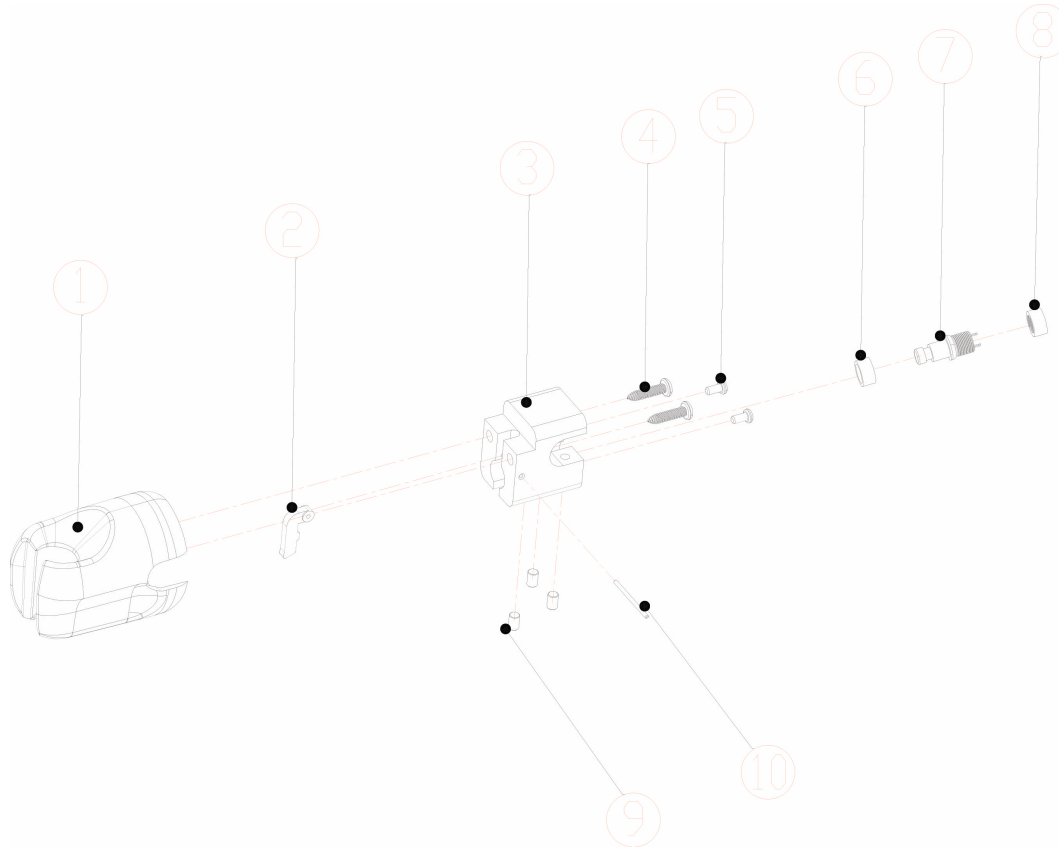


No.	Description	Part Number
6	Switch Spacer	OMC2375-RP4-6
7	Electric Switch	OMC2375-RP4-7
8	Switch Spacer	OMC2375-RP4-8

Reference Parts list on page 1 for complete HP Holder part numbers. This list is only for Switch And spacer assemblies.



### MC3600 Parts List

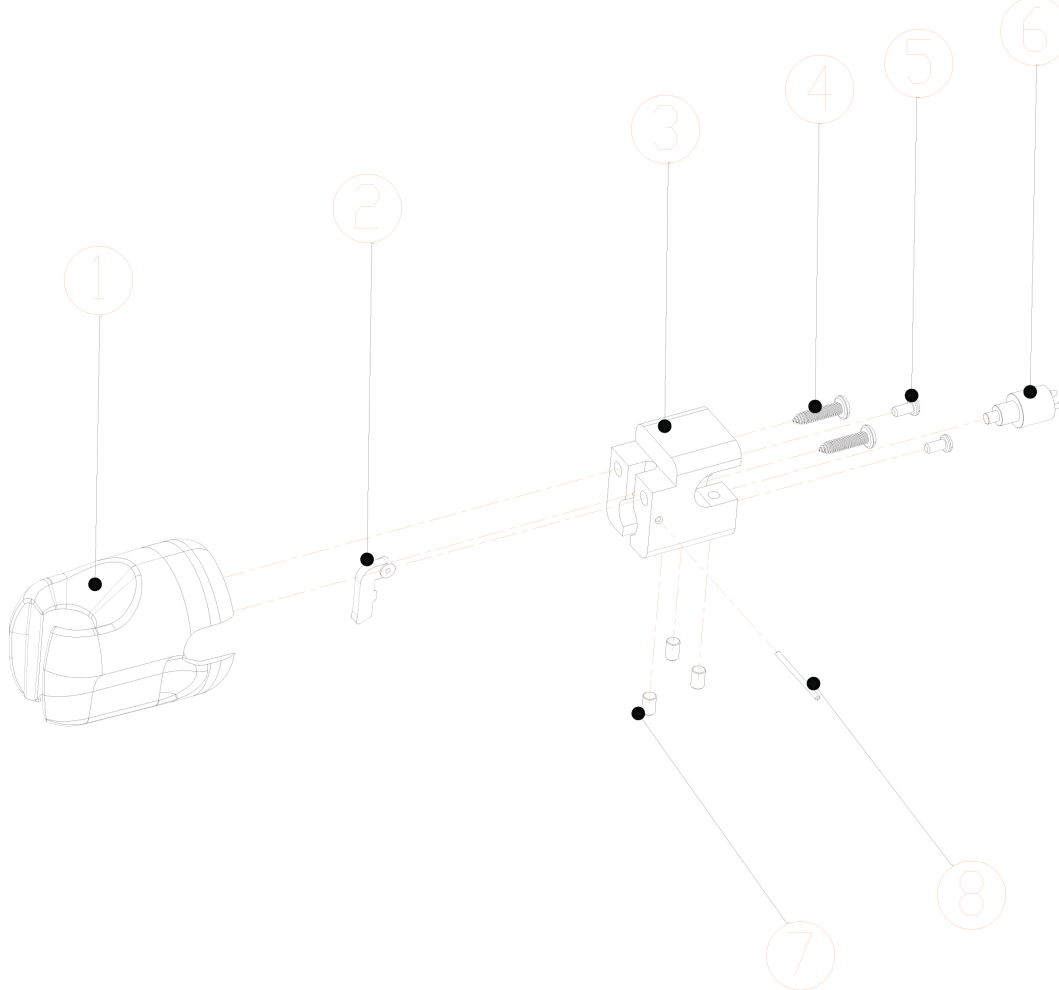

No.	Description	Part Number
6	Air Switch HP holder	OMC2375-RP5-6

Reference Parts list on page 1 for complete HP Holder part numbers. This list is only for Switch And spacer assemblies.